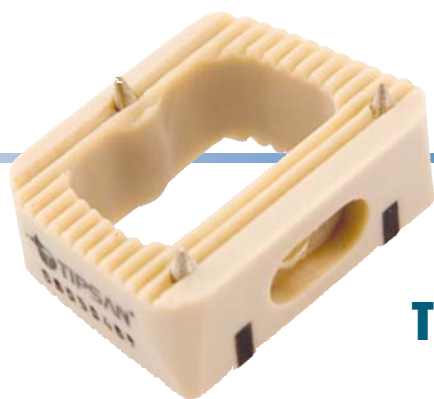
**CERVICAL
PEEK CAGE**

**CERVICAL PEEK CAGE
SERVIKAL PEEK KAFES**

The key in

orthopaedic

sciences

**SURGICAL TECHNIQUE
IMPLANTS & INSTRUMENTS**

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Introduction

Giriş

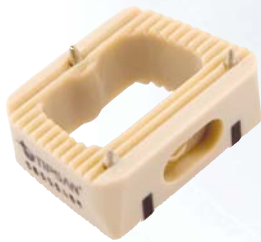
EN Anterior arthrodesis of the cervical spine is an indication for the Cervical PEEK Cage. The use of cages supports the reestablishment of the disc height and opening of the neuralforamen.

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Material Malzeme

EN Manufactured from PEEK according to ASTM F 2026. It is selected for its; Radiolucency is given by the PEEK-OPTIMA material, a poly-ether-ether-ketone whose durability and flexibility are similar to those of cancellous and cortical bone, with the high level of biocompatibility needed for long-term implantation in the body. Moreover, the material is highly transparent to x-rays and does not generate radiographic distortions, thus permitting reliable positional verification of the implant.

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Please Note : This document is intended as a guide for the surgeon only. There are multiple techniques for the insertion of Cervical PEEK Cage and, as with any surgical procedure, a surgeon should be thoroughly trained and beware that this procedure is appropriate for the patient before proceeding.

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Distraction Distraksiyon

EN Once the target area is confirmed with the image intensifier, Caspar Spreader Screws (Ref: 10604321005) are screwed in with the Caspar Spreader Screw Driver (Ref: 10604320001), in the vertebrae above and below the disc to be removed.

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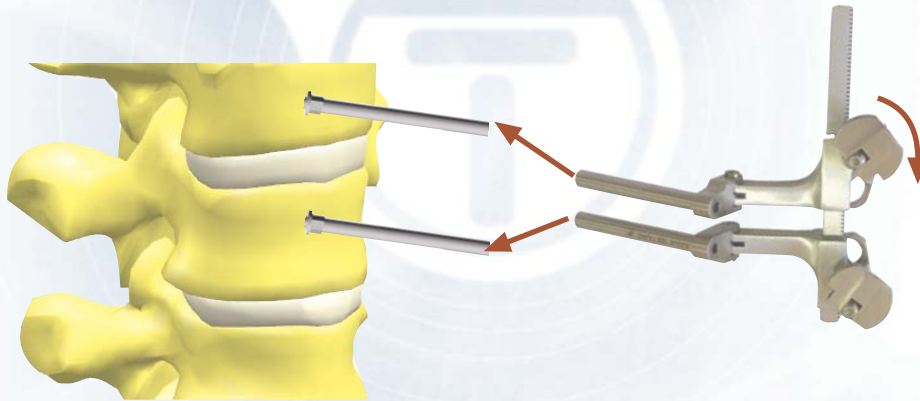
EN The sleeves of the Caspar Spreader (Ref: 10603060001) are inserted onto the screws. Distraction of the vertebrae is provided by turning the wing nut clockwise.

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EN Depending on the pathology to be treated, a single or multi-level discectomy can be carried out.

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Discectomy and Decompression Diskektomi ve Dekomprasyon

EN A complete discectomy should leave the adjacent vertebral endplates cleared from all the disc material and the cartilaginous layer.

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Cervical PEEK Cage

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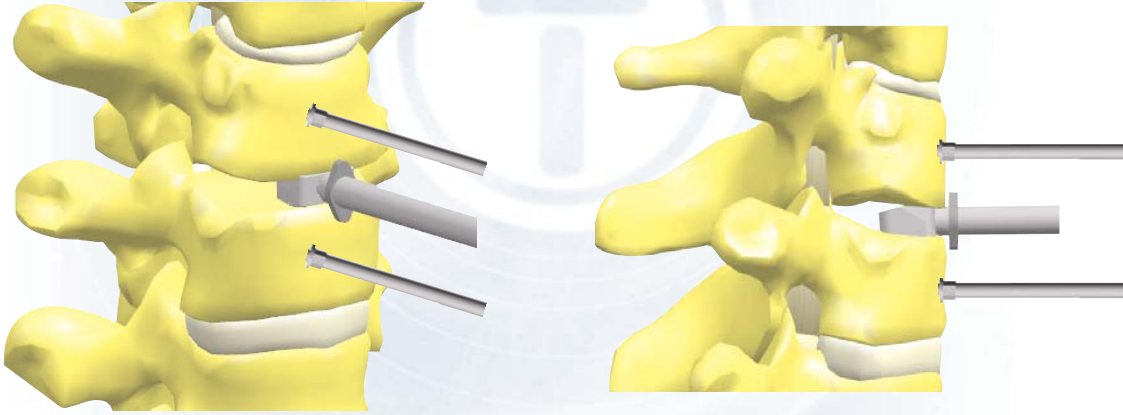
EN The appropriate Cervical PEEK Cage trial cage is connected to the Cervical cage holder (Ref: 10607110011) and the safety stop is adjusted.

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EN The trial is then introduced into the intervertebral space under fluoroscopy. Ensure the handle is positioned in the "cranial" position. The size of the trials correspond to the Cervical PEEK Cage.

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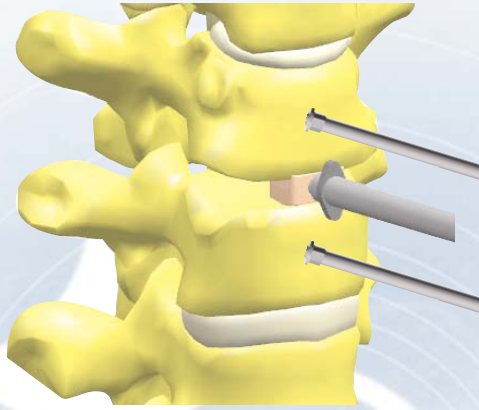
EN The selected Cervical PEEK cage is connected to the Cervical cage holder and placed on the Socket.

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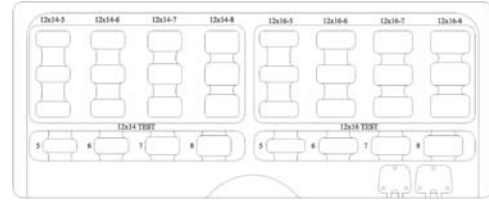
Cervical PEEK Cage

Notes/Notlar

The products being manufactured by TIPSAN A.S. has been certified from SZUTEST (Strojirensk• Zkušební Ústav) for EC certificate **(CE 1015)** according to Annex II. 3 and II.4 of Council Directive 93/42/EEC concerning medical devices

Kemalpasa Cd. 7404/1 Sk. No: 3 Pinarbasi - IZMIR - TURKEY
Tel: +90 232 479 56 54 Fax: +90 232 479 58 27
marketing@tipsan.com.tr www.tipsan.com.tr