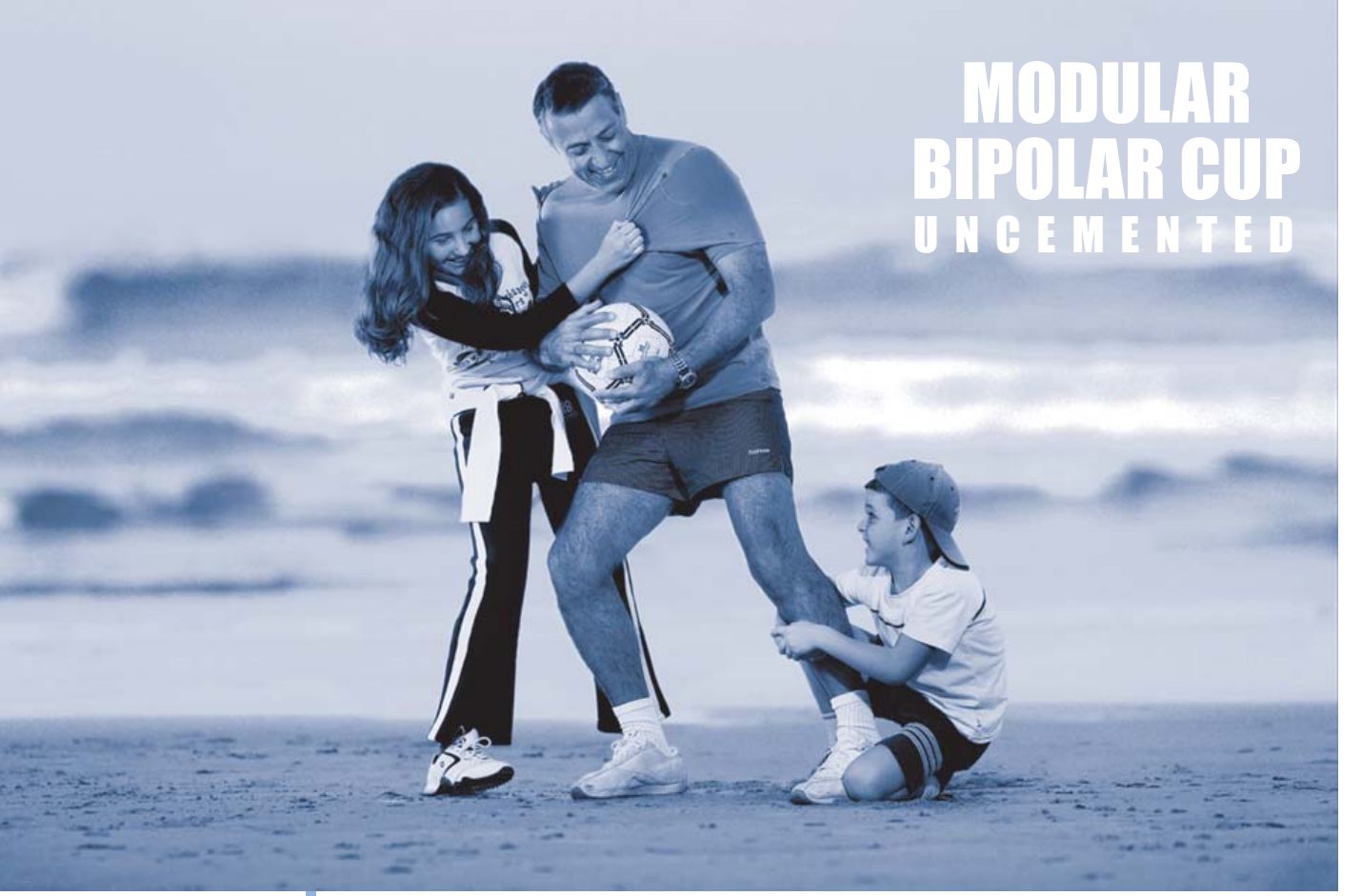
**MODULAR  
BIPOLAR CUP  
UNCEMENTED**

**MODULAR BIPOLAR CUP  
UNCEMENTED  
MODÜLER BİPOLAR KAP SEMENTSİZ**

**The key in**

**orthopaedic**

**sciences**

**SURGICAL TECHNIQUE  
CERRAHİ TEKNİK  
IMPLANTS & INSTRUMENTS  
İMLANTLAR VE EL ALETLERİ**

## **CONTENTS** **İÇİNDEKİLER**

**Introduction**  
Giriş

2

**Surgical Steps**  
Cerrahi Adımlar

3

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## Introduction

## Giriş

**EN** Placement of cup together with a femoral head on stem gives an articulated partial prosthesis.

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Material  
Malzeme

**EN** PE-UHMW crosslinked material meets ASTM F 648 and ISO 5834-1/2 standards. Selected for its, high degree of purity, good biotolerance, good mechanical performance and friction properties.

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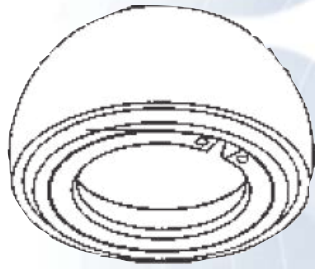
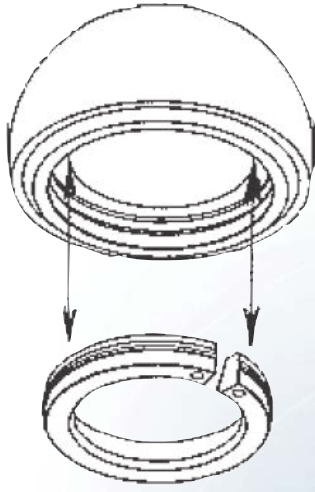
**EN Please Note :** This document is intended as a guide for the surgeon only. There are multiple techniques for the insertion of Modular Bipolar Cup and, as with any surgical procedure, a surgeon should be thoroughly trained and beware that this procedure is appropriate for the patient before proceeding.

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## Surgical Steps Cerrahi Adımlar

**EN** After removal of the femoral head, the head diameter can be measured to determine the approximate size. The appropriate sized trial head can be placed into the acetabulum to determine fit. The proper size has been determined when the trial has a snug fit in the acetabulum. Trial reduction to determine fit and range of motion can be performed with either the broach or femoral implant in place.

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**EN** Once the appropriate size bipolar implant has been selected, it can be assembled. After opening the bipolar from the sterile package, the locking ring is removed with the Bipolar Cup Clamp (Ref:10601271101). The ring clamp is placed into the two holes on the locking ring and compressed to release the ring from the bipolar shell.

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- EN** Insert the ring around the taper; the ring will be deformed by elasticity to pass through the femoral head and the two holes on the ring have to be oriented towards the stem's metaphyseal part.

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**EN** Set up the bipolar cup, whose diameter has been chosen during preoperative planning, in the acetabular cavity. Set the joint.

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## Attention/Dikkat

**EN** A bipolar cup is used without any acetabular cavity preparation.

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**EN** Set up the ring. Set up the ring manually in order to get the "clic" for device locking .

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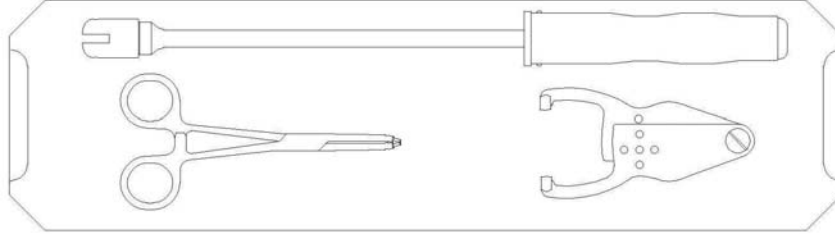
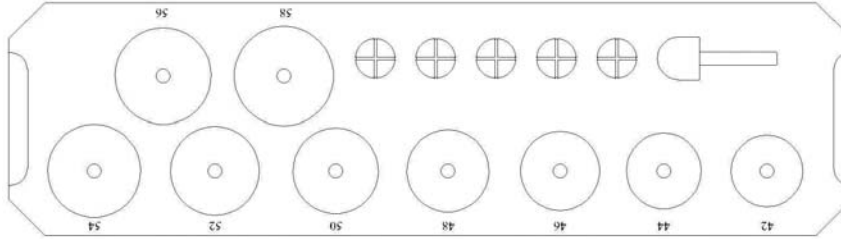
**EN** If required, the bipolar head can easily be removed with the bipolar ring clamp. The ring clamp is placed into the two holes on the locking ring and compressed, releasing the locking ring so that the bipolar head can be removed.

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## Notes/Notlar

---

The products being manufactured by TIPSAN A.S. has been certified from SZUTEST (Strojirensk•  
Zkušební Ústav) for EC certificate **(CE 1015)** according to Annex II. 3 and II.4 of Council Directive  
93/42/EEC concerning medical devices

Kemalpasa Cd. 7404/1 Sk. No: 3 Pinarbasi - IZMIR - TURKEY  
Tel: +90 232 479 56 54 Fax: +90 232 479 58 27  
marketing@tipsan.com.tr www.tipsan.com.tr