**MODULAR STRAIGHT STEM  
UNCEMENTED**

**MODULAR STRAIGHT  
STEM UNCEMENTED**

**The key in**

**orthopaedic**

**sciences**

**SURGICAL TECHNIQUE  
CERRAHI TEKNİK  
IMPLANTS & INSTRUMENTS  
İMLANTLAR VE EL ALETLERİ**

## CONTENTS İÇİNDEKİLER

### Introduction

Giriş

2

### Femoral Neck Osteotomy

Femoral Boyun Osteotomisi

4

### Accessing the Femoral Canal

Femoral Kanalin Açılması

4

### Contouring the Stem Envelope

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5

### Trial Reduction

Test İmplantının Denenmesi

5

### Stem Insertion

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6

### Head Insertion

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7

### Instruments&Trays

El Aletleri&Tavalar

8

### Combination Chart

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10

## Introduction

### Giriş

**EN** It is a uncemented total hip prosthesis. It can be used for all the classical indications of total hip arthroplasty such as primary or secondary hip osteoarthritis, rheumatic coxitis, avascular necroses and broken femoral neck. Neither age osteoporosis nor modifications to uncemented implants of this type. This prosthesis is optionally available with hydroxapatite coating (HA). Its HA coating uses the osteo-conductive qualities of this efficient biomaterial, which reproduces the mineral fraction of natural bone.

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**EN** The fixation of the stem in the proximal area should guarantee that transmission of the force is retained within this area. the interposition of the cancellous structure reduces the danger of the bond between bone and prosthesis becoming stiff. On introduction, the axial ribs exercise a cutting action and promote stability. In order to avoid tension peaks, the surface of the distal part of the stem are well rounded. The shape of distal part of the stem is kept relatively slender, thus direct contact with the cortex is where possible avoided. This means greatly reduced incidence of pain for the patient, plus the trabecular structures can form between the cortex and implant, which eventually result in osseointegration.

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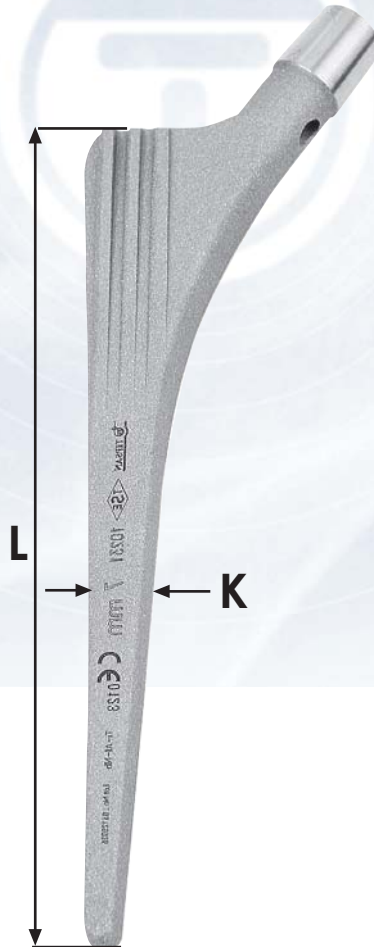
**Please Note :** This document is intended as a guide for the surgeon only. There are multiple techniques for the insertion of Modular Straight Stem Uncemented and, as with any surgical procedure, a surgeon should be thoroughly trained and beware that this procedure is appropriate for the patient before proceeding.

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## Femoral Neck Osteotomy Femoral Boynun Osteotomisi

**EN** The osteotomy is normally carried out parallel to the intertrochanteric line. The axis of the prosthesis and the large proximal fins must coincide with the axis of the femur. Because of the normal anteversion of the femoral neck, the prosthesis can only be positioned in the correct axial direction if the length of neck stump is medially no more than 1 cm.

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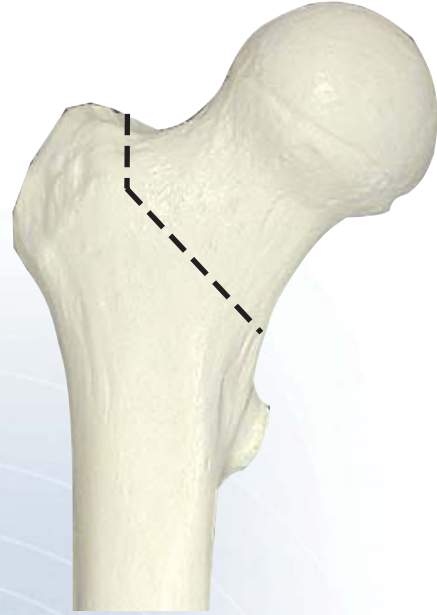
**EN** A very important step is the trapezoidal opening of the medullary cavity. This opening, which may be made with a Tuke saw, should reach the tip of the greater trochanter. Then the femoral head is removed using a Femoral Head Extractor (Ref: 10606100001). The resected cancellous bone is removed with a gouge to expose the medullary canal.

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
**EN** The awl (Ref:10610331003) is introduced in the region of the resected tip of the greater trochanter and should follow the long axis of the femur. It will orientate itself to the course and shape of the medullary canal and provide a guideline for the ideal seating of the stem.

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## Contouring the Stem Envelope Raspalama Aşaması

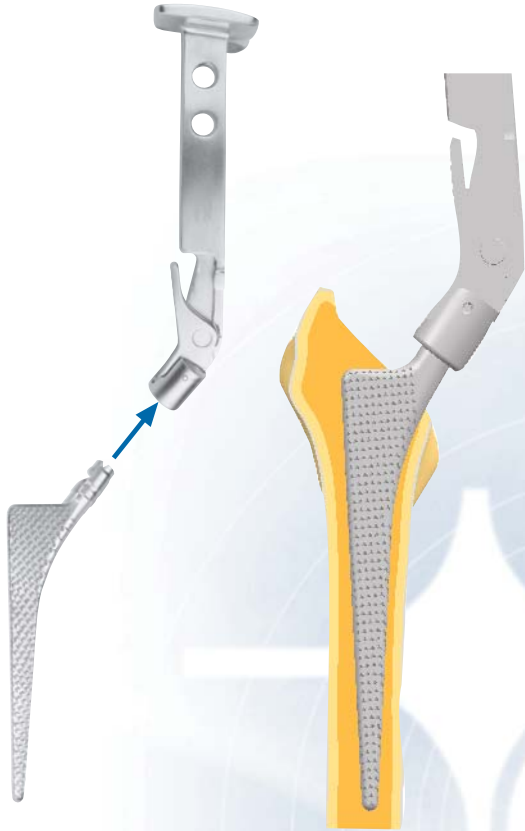**EN** The final seating of the prosthetic stem is prepared using the sizing Rasps and Modular Rasp Handle (Ref: 10610011050), always beginning with the smallest one. To achieve a tight proximal press-fit avoiding a varus or valgus position, the rasps are introduced in the same place and in the same direction as the awl. Great care must be exercised and rotational movements avoided. The rasps are driven into the predetermined depth, taking into account the desired antetorsion of 10° to 15°.

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**EN** After the last rasp has been impacted, there should be a layer of mechanically compressed cancellous bone reaming between the rasp and the cortical bone -particularly in the medial section- with an ideal thickness of 3-5 mm or more. The last rasp is left in the femur for using it as trial prosthesis

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**EN** A test head is fitted to the neck by pushing it onto the taper. To remove a test head, simply pull it.

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**EN** The stability, range of movement and the muscular tension are checked.

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**EN** The test head and rasp are removed.

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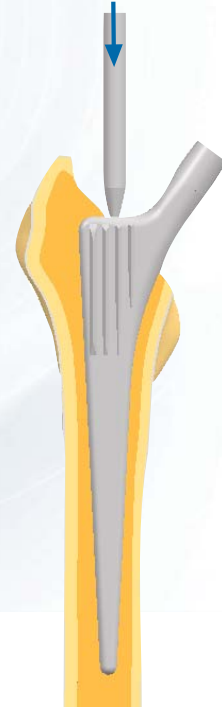
**EN** After removal of the rasp, a prosthesis of the appropriate size (as same size as last used raps) is inserted and driven in with using rasp bar (Ref: 10607031321) until it is completely stable. In this process, it is important to proceed with the necessary light touch. This is learned with experience. It should be remembered that because of the wedge mechanism, an excessive load may be exerted that can bear on the trochanter to the point that it might cause a fracture.

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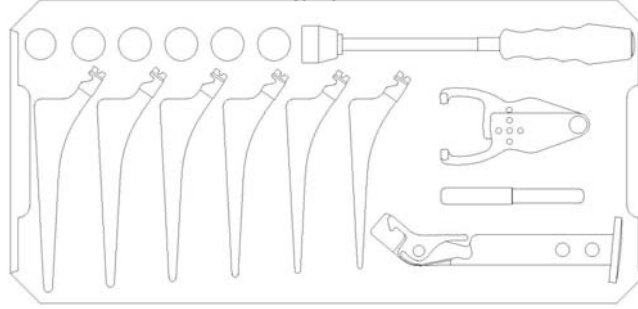
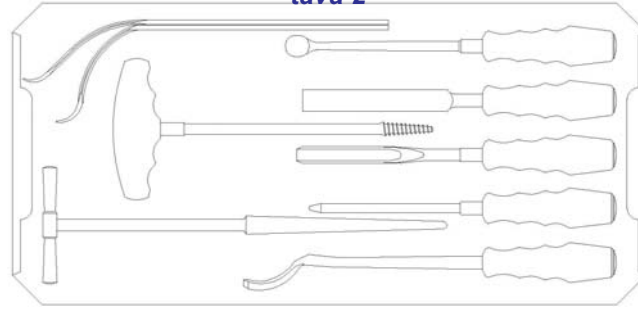
Femoral Head Extractor  
Tirbişon

10607031321

Impactor & Extractor for TMC-3  
Çakma Pimi

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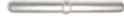
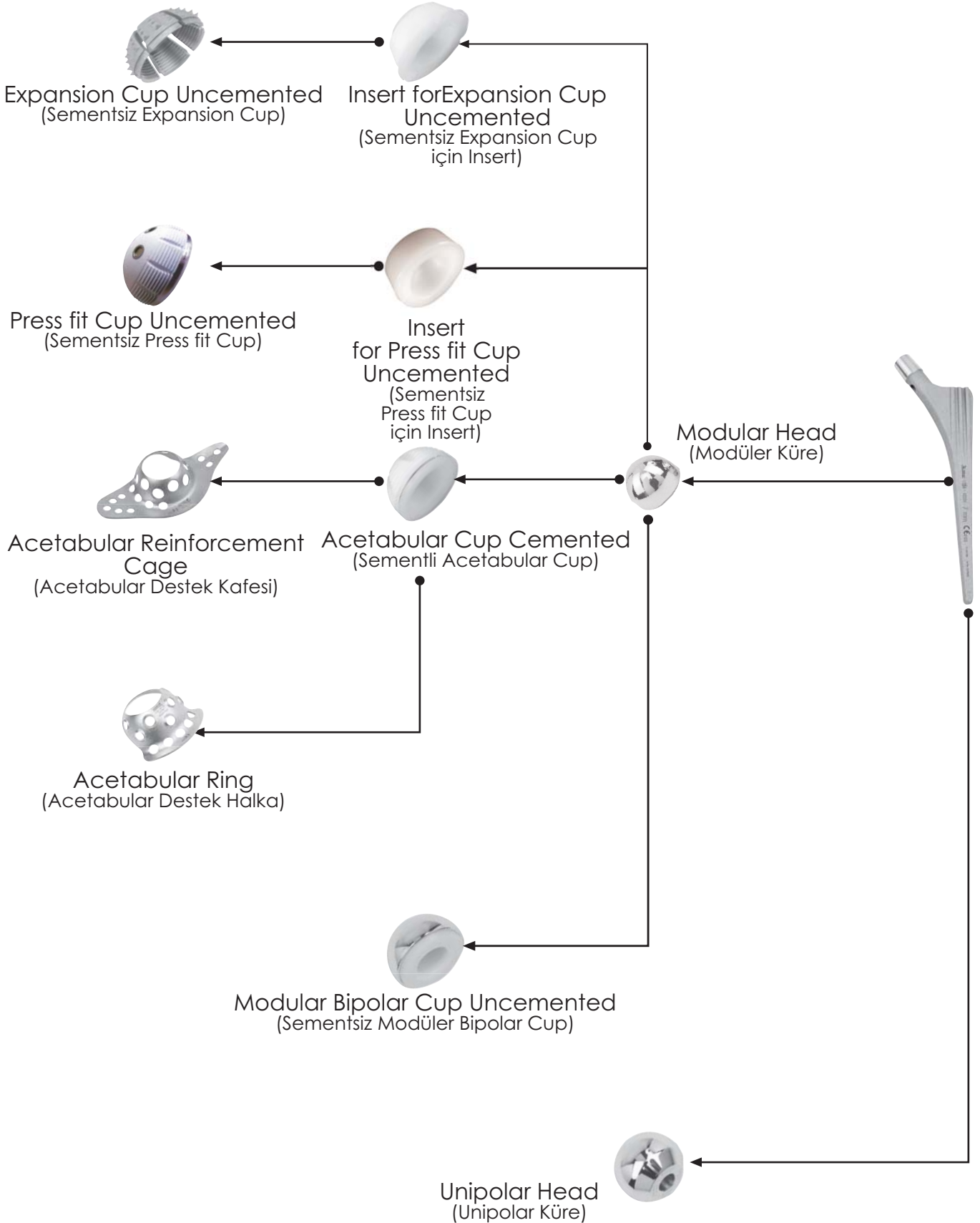
The products being manufactured by TIPSAN A.S. has been certified from SZUTEST (Strojirensk•  
Zkušební Ústav) for EC certificate **(CE 1015)** according to Annex II. 3 and II.4 of Council Directive  
93/42/EEC concerning medical devices

Kemalpasa Cd. 7404/1 Sk. No: 3 Pinarbasi - IZMIR - TURKEY  
Tel: +90 232 479 56 54 Fax: +90 232 479 58 27  
marketing@tipsan.com.tr www.tipsan.com.tr